

7-5-2 OPERATION LIST FOR MAC OS

	Mouse	Keyboard
Help	Main menu > Help	[F1]
Open Slide	Main menu > Browse	[O]
Exit Slide	Tab area > X	[Ctrl]+ [W] [ESC]
	Right-Click in the tab area > Close	
Exit Software	Upper-right of the screen > X	[Command] + [Q]
	Main menu > Exit	
Tool widget *1	—	[T]
Move Slide	Left-Drug	[←], [→], [↑], [↓]
	Left-Click (in Map image)	
	Joy stick widget	
Magnification	Mouse wheel	[+],[−], [1]~[8] *2 [1] =1.25x [5] =20x [2] =2.5x [6] =40x [3] =5x [7] =80x [4] =10x [8] =160x
	Right-Drug (allow in Map image)	
	Double-Click	
	Lens widget	
Focus Plane	[Ctrl] + Mouse wheel	[Page Up],[Page Down]
Rotation	View widget	[R] each 90 degree [Option] + [←], [→]
Mirror Image	Mirror widget	[Option] + [M]
Flip Image		[Option] + [F]
Reset Image	—	[Space] , [0]
Split Screen	Right-Click in the tab area	—
Synchronize	Split menu > Synchronize	[Sift]+ [S]
Desynchronize	Split menu > Desynchronize	[Sift]+ [S]

	Mouse	Keyboard
Close window	—	[Esc]
Multi Window	Main menu > New Window	[Command] + [N]
Synchronize all	Main menu > Synchronize All	—
Desynchronize all	Main menu > Desynchronize All	—
Show/Hide Annotation	—	[A]
Show/Hide Annotation list	—	[L]
Add Annotation	Rite-Click Menu > Select Annotation type	—
Go to Annotation	Right-Click on the Annotation > Annotation menu > Go To	—
Edit Annotation	Right-Click on the Annotation > Annotation menu > Edit	—
Move Annotation	Right-Click on the Annotation > Annotation menu > Move	—
Copy Annotation	Right-Click on the Annotation > Annotation menu > Copy	—
Cut Annotation	Right-Click on the Annotation > Annotation menu > Cut	—
Delete Annotation	Right-Click on the Annotation > Annotation menu > Delete	—
Paste Annotation	Annotation menu > Paste	[Ctrl] + [V] [Command]+ [V]
Export Annotations	Annotation menu > Export	[Option] + [E]
Enable Tracking	Map menu > Enable Tracking	—
Show Track Map	Map menu > Show Track Map	—
Reset Track Map	Map menu > Reset Track Map	—
Bird's Eye View	—	[W]
Map Image	Rite-Click Menu > Export Image >Show/Hide Map	[M]
Scale Bar	Rite-Click Menu > Export Image >Show/Hide Scale bar	[B]
Change Scale Bar types	—	[Ctrl] + [B] [Command]+ [B]

	Mouse	Keyboard
Glass Slide image	Main menu > Slide Image	[G]
Export view	Main menu > Export Image	[E]
Export Image	Rite-Click Menu > Export Image	[Ctrl] + [E] [Command]+ [E]
Copy Image	Main menu > Copy to Clipboard	[C]
	Rite-Click Menu > Copy to Clipboard	[Ctrl] + [C] [Command]+ [C]
Screenshot	Main menu > Take Screenshot	[Command]+[Shift]+[3]
Settings	Main menu > Settings	[P]
Image Control	Main menu > Image Controls	[I]
Image Information	Main menu > Image Info	[F]
Move to the next tab Back to the previous tab	—	[Ctrl]+[Tab] [Ctrl]+ [Shift]+[Tab]
Open the next slide Open the previous slide	—	[Tab] [Shift]+[Tab]

*1: Lens widget and Focus widget.

*2: Lens menu and shortcut key are different depend on the scan mode when you take the image.

Note

- Please refer each chapter for details.